

# Oxygen Concentrator Warm Up Instructions

Stationary concentrators should go through a 10-Minute warm-up period each time they are turned on. No supplies should be attached to the concentration during the warm-up.

## **Warm Up Steps:**

1. Turn the flow setting on the concentrator to the max and leave it running for a minimum of 10 minutes.
    - Do not use the device during this time.
  2. Make sure the concentrator is at least 1 ft away from any walls, curtains, or furniture.
  3. Leave running for 10 minutes.
- If the machine is fully functional the Red and Yellow lights will turn to Green after the warm up period.

Thank you!